Power wheelchair maintenance hands-on activity checklist

Maintenance activities	Activity done	Mark if Not applicable	Frequency
Rear wheels			
Inspect tires			Monthly
If pneumatic tires, inspect tires' pressure			Weekly
Casters	1	•	•
Inspect caster bearings			Monthly
Clean caster bearings			Monthly
Inspect anti-tip			Monthly
Cushion			
Inspect cushion and cover			Weekly
Wipe down cushion and cover			Monthly
Supports	1	ı	1
Inspect foot support			Monthly
Inspect arm support			Monthly
Inspect leg support			Monthly
Inspect back support			Monthly
Inspect other supports (thigh, trunk)			Monthly
Inspect positioning belts			Monthly
Wipe down supports			Monthly
Frame	1	1	•
Inspect welds			Monthly
Inspect shrouds			Monthly
Inspect suspension			Monthly
Wipe down frame			Monthly
Electrical System			
Inspect motor disengage levers			Daily
Inspect motor noise			Daily
Inspect driving and braking			Daily
Inspect power seat functions			Daily
Clean power seat functions mechanism and track			Monthly
Inspect controller, joystick, other controls, indicators, and horn			Daily
Inspect wiring and electronic connections			Monthly
Inspect battery and charger			Monthly
Charge the battery			Daily